

schülke -+



Alkaline liquid cleaner for the automated reprocessing of medical instruments and laboratory accessories

thermosept® alka clean forte

Our Plus

- alkaline instrument cleaner, hence not aggressive to materials and suitable for anodized aluminium
- minimizes risk of iatrogenic transfer of CJD/vCJD (acc. to RKI report)
- powerful cleaning
- broad range of applications

Please note recommendations of machine and instrument manufacturers.



Application areas

thermosept® alka clean forte is particularly suitable for the efficient cleaning of surgical and microsurgical instruments, rigid endoscopes, anaesthetic accessories, containers and laboratory accessories. thermosept® alka clean forte is suitable for use in washer/disinfectors and multichamber washers/disinfectors.

Instructions for use

Dosage:

Standard recommendation: 0.5 % (5 ml/l).

Depending on level of soiling: 0.3 - 1 % (3 - 10 ml/l) may be used. Dosing is performed by machine-integrated dosing pumps.

It is recommended that the water used for cleaning and the following program steps be at least softened and ideally demineralized (as for thermal disinfection). Other individually customizable program operations are available for the treatment of eye instruments or for program variations. Combination with thermosept® NKZ or NKP ensures an optimal treatment result. Neutralization is not essential if water of suitable quality is used. Exclusively demineralized water should be used for thermal disinfection.



thermosept® alka clean forte

Product data

Composition:

Labelling according to Regulation (EC) No. 648/2004: < 5 % nonionic surfactants, < 5 % polycarboxylates, < 5 % anionic surfactants, < 5 % NTA (nitrilotriacetic acid) and salts thereof, enzymes

Other ingredients: solubilizer, corrosion inhibitor

Chemical-physical data

Color	nearly colourless
Density	ca. 1,08 g/cm ³ / 20 °C / 1.013 hPa
Flash point	ca. 50 °C / Method : DIN 51755 Part 1
Form	liquid
pH	ca. 12,5 / 20 °C / concentrate
Viscosity, dynamic	ca. 5 mPa*s / 20 °C / Method : ISO 3219

Accessories

Accessories	Item no.
5 l/10 l-Canister key	135810
Canister opener 30 l	135901

Related Products

- thermosept® NKP
- thermosept® NKZ

Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining out high quality standards.

Hazard notes and safety advice

Xi	Irritant
R36/38	Irritating to eyes and skin.
S26	In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

Expert opinion and information

Please visit our website for an overview of all available literature/reports on the product: <http://www.schuelke.com/>.
For individual questions: Customer Care Phone: +49 40 52100-666 E-Mail: info@schuelke.com

Special advice

Do not mix concentrate or working solution with acids.

Information for order

Item	Delivery form	Item no.
thermosept alka clean forte 200 kg FA	1/drum(s)	on request
thermosept alka clean forte 5 kg KA	1/Canister	on request
thermosept alka clean forte 10 kg KA	1/Canister	on request
thermosept alka clean forte 30 kg KA	1/Canister	on request

These products are not available in every country. For more information please contact our local subsidiary or distributor.



schülke Headquarters
Schülke & Mayr GmbH
Robert-Koch-Str. 2
22851 Norderstedt, Germany
Phone +49 (0) 40 - 52100 - 0
Fax +49 (0) 40 - 52100 - 318
www.schuelke.com
mail@schuelke.com

Schülke & Mayr UK Ltd.
1 Jenkin Road
GB-Sheffield S9 1 AT
Phone +44 (0) 1142 - 5435 - 00
Fax +44 (0) 1142 - 5435 - 01
mail.uk@schuelke.com

Schülke & Mayr Ges.m.b.H.
Seidengasse 9
1070 Wien, Austria
Phone +43 (0) 1 - 5232501-0
Fax +43 (0) 1 - 5232501-60
office.austria@schuelke.com



A company of the Air Liquide-Group