



Acidic softener components for the thermal reprocessing of bedpans and urine bottles.

thermosept® SKS

Our Plus

- prevents limescale
- low foaming formular
- good material compatibility (suitable for aluminium and plastic)

Application areas

thermosept® SKS is suitable for the thermal repossessing of bedpans and urine bottles in any commercially available equipment. Product is applicable for all levels of water hardness.

Product Profile

thermosept® SKS is a liquid, acidic softener (concentrate), free of surfactants.

Instructions for use

Dosage:

0.01 % per 1° dH (corresponds to 0.18 mmol/l water hardness), generally that corresponds to a dose of 0.1 - 0.3 % (1 - 3 g/l water).

Dosing is performed by machine-integrated dosing pumps. Please check dosing pumps for acid resistance prior to application.

If there is a large amount of limescale on the heating element descaling is recommended before using thermosept® SKS. thermosept® SKS prevents limescale in the steam generator, also with hard water. To avoid limescale in general, not only should regular maintenance be carried out but the canister should be replaced at regular intervals.



Product data

Composition:

hardness stabiliser, corrosion inhibitors, organic acids

Chemical-physical data

Color	colourless
Density	ca. 1,08 g/cm ³ / 20 °C
Flash point	> 100 °C
Form	liquid
pH	ca. 1,3 / 20 °C / concentrate
Viscosity, dynamic	no data available

Information for order

Item	Delivery form	Item no.
thermosept SKS SOKA 5 kg KA	1/Canister	on request
thermosept SKS 5 kg KA	1/Canister	on request

These products are not available in every country. For more information please contact our local subsidiary or distributor.

Accessories

Accessories	Item no.
5 l/10 l-Canister key	135810

Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining out high quality standards.

Expert opinion and information

Please visit our website for an overview of all available literature/reports on the product: <http://www.schuelke.com/>.
For individual questions: Customer Care Phone: +49 40 52100-666 E-Mail: info@schuelke.com



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.



schülke Headquarters
Schülke & Mayr GmbH
Robert-Koch-Str. 2
22851 Norderstedt, Germany
Phone +49 (0) 40 - 52100 - 0
Fax +49 (0) 40 - 52100 - 318
www.schuelke.com
mail@schuelke.com

Schülke & Mayr UK Ltd.
1 Jenkin Road
GB-Sheffield S9 1 AT
Phone +44 (0) 1142 - 5435 - 00
Fax +44 (0) 1142 - 5435 - 01
mail.uk@schuelke.com

Schülke & Mayr Ges.m.b.H.
Seidengasse 9
1070 Wien, Austria
Phone +43 (0) 1 - 5232501-0
Fax +43 (0) 1 - 5232501-60
office.austria@schuelke.com